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De DPT LICITACAO <licitacao@abamedicalgo.com>

Para <pregao@senadorcanedo.go.gov.br>

Data 17/12/2025 15:10

 Pe 106.2025 SENADOR CANEDO.rar (~11 MB)

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04.369.147/0001-04

(62) 9 8318-0162

INSTRUCTIONS FOR USE OSTOMY POUCHES

Ostomy One-Piece Pouch - Reg:1022320011

TYPE POCHE	POUCHES / BOX	CLAMPS	STOMA METER
Drainable pouch	10 or 15	2	1
Closed pouch	20	**	1
Urostomy pouch	10	**	1

Ostomy Two-Piece Pouch - Reg:1022320017

TYPE POCHE	POUCHES / BOX	CLAMPS	STOMA METER
Drainable pouch	10	2	1
Closed pouch	10	**	1
Urostomy pouch	05	**	1

*Box with 5 units.

DESCRIPTION FOR ONE-PIECE POUCH

CASEX Colostomy and Ileostomy pouches are made with three layers film, EVA/PVDC/EVA, opaque or transparent and synthetic resin, composed by sodium carboxymethylcellulose, pectin, gelatin and polyisobutylene. It also has a protective skin barrier, microporous adhesive (optional) and activated carbon filter (optional for drainable pouches). CASEX Urostomy pouch, besides the information above, has an anti-reflux valve, nozzle to drain the pouch (filter is not applicable).

DESCRIPTION FOR TWO-PIECE POUCH

Casex Two-Piece Pouch has the same items of one-piece pouch adding a base/wafer and plastic flange in order to affix it on the pouch.

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Colostomy/Ileostomy pouches are used to collect excreta. Urostomy pouches are used to collect urine.

APPLICATION FOR ONE-PIECE POUCH

1. Measure the stoma size with the scale, which comes attached to the carton box;
2. Before cutting the resin (adhesive with circles), separate the front side from the backside of the pouch, in order to avoid cutting the front side;
3. Cut the initial hole slightly larger than the diameter of the stoma following the scale printed on the resin paper;
4. Remove the paper from the resin;
5. Do not remove the microporous adhesive paper around the resin at this moment;
6. Press the resin against the skin carefully, ensuring that it stays flat and unwrinkled.
7. Remove the microporous adhesive paper around the resin and smooth it over the skin. The pouch can stay applied while it remains comfortable and safe.

APPLICATION FOR TWO-PIECE POUCH

After the steps above, attach the pouch to the base/wafer starting from the bottom and continuing with a slight pressure to the perfect fit. If necessary, use Casex elastic belt for better fixation.

APPLICATION FOR ANATOMIC CLAMP FOR DRAINABLE POUCHES

- Hold the clamp opened;
- Put the pouch on the blade;
- Close the clamp until lock the blade;
- To open, hold the body clamp and push the blade out.

DRAINAGE FOR UROSTOMY POUCHES

- Open the valve and let the liquid drain.
- Close the valve once the pouch is empty

PRECAUTIONS

Keep away from children. Maintain in a dry and cool place. Do not use the product after the expire date. The batch number and the expire date are printed on each package and on each product. In case of skin irritation, suspend the usage and follow medical orientation.

CONTRAINDICATIONS

usage on people with known sensibility to pouch adhesives or components. The use of moisturizers and emollients affect the product adherence.

Store at room temperature. Avoid refrigeration and exposure to excessive humidity.

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