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Use Dental Air Compressor oil free, with working pressure between 5.5 and 7.0 bar (80 and 100 PSI), minimum airflow of 150 l/min and a 30 liters tank.

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The power supply system must be single-phase and specific grounding.

Use RC Circuit Breaker 10A 30mA exclusive for the dental unit. That circuit breaker shall be located near to the equipment, with free access to the user.

If the power grid presents voltage variation, the installation of a surge protection device is recommended.

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Voltage supply: 118/127/220/230 V.
(Equipment tension selection necessary)
Frequency: 50/60 Hz, automatic set.
Equipment Inlet Air Pressure: from 5.5 to 7 bar.
Water Pressure: between 2.8 and 4 bar.
Water Consumption: 3.5 l/min.

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Before undertaking the dental office project check the legislation in force in your region, including electrical and hydropneumatic normative guidelines.

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This template contains instructions for the preparation of the connections and spaces for installation of your Dental Set. Please contact your nearest authorized service or our After Sales Department by the e-mail export3@olsen.odo.br for any doubts you may have.

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TEMPLEATE AND CONNECTION PLANT FOR INSTALLATION OF DENTAL UNITS

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